


Canestra Hip Length Gauge

Designed by Vince Canestra, MD

Helps determine leg length and hip offset in total hip arthroplasty, including minimally invasive techniques

Set consists of one Ruler, one Pin Insertor/Extractor Handle, one 100 mm Pin, one 130 mm Pin (not shown), and a sterilization case (not shown).

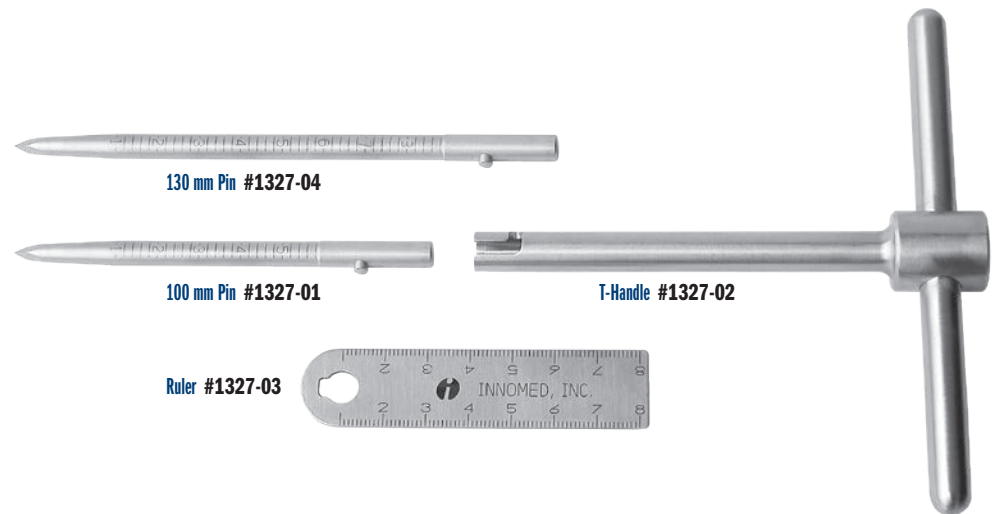

PRODUCT NO'S:

1327-00 [Set]

Set Includes / Replacement Parts:

1327-01 [Pin – 100mm]

1327-02 [T-Handle]
Dimensions: 8" x 5" (20,3 cm x 12,7 cm)

1327-03 [Ruler]

1327-04 [Pin – 130mm]

1025 [Sterilizable Case]



* Accuracy Statement for devices with a measuring function:

Devices with a measuring function are considered non-critical, non-calibrated, and are intended for reference-only measurements. While it provides a helpful visual guide for relative measurements, it is not calibrated to specific precision standards. Measurements taken with this device should not be considered accurate enough for critical applications requiring strict dimensional tolerances. General manufacturing tolerances of $\pm 1\%$ at each graduation, or 2% of the full scale apply, but no conformance qualifications are provided or implied.

— over —

www.innomed.net



ISO 13485:2016

FREE TRIAL ON MOST INSTRUMENTS



103 Estus Drive, Savannah, GA 31404
info@innomed.net

912.236.0000 Phone
912.236.7766 Fax

Innomed-Europe Tel. +41 41 740 67 74
info@innomed-europe.com


1.800.548.2362

INNOMED

Canestra Hip Length Gauge

Designed by Vince Canestra, MD

Helps determine leg length and hip offset in total hip arthroplasty, including minimally invasive techniques

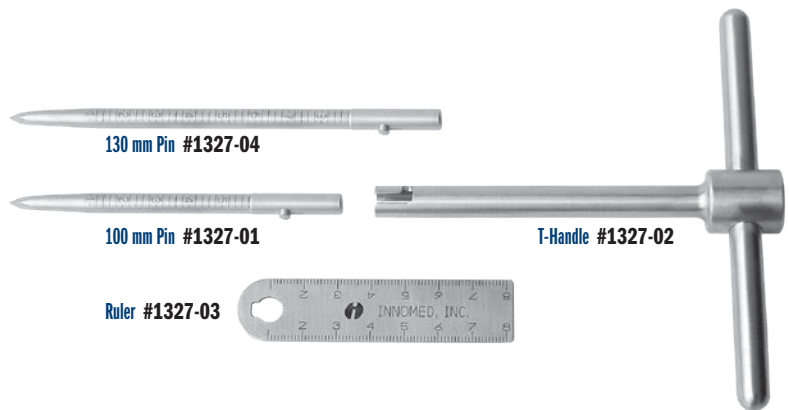

After incising the hip capsule prior to dislocating the hip, the graduated bone pin is inserted just proximal to the acetabular roof from the lateral direction ("A" in Figs. 1 & 2).

The legs are aligned together so that the patellae and heels are collinear and so that the position can be duplicated later in the course of the procedure.

The ruler is inserted over the bone pin until it lays flush on top of or level with the vastus tubercle of the greater trochanter. (Figs. 1 & 2)

The initial hip offset is noted from the markings on the bone pin at the point it intersects with the ruler (Fig. 3). A reference point or mark is made along the ruler as it lays on the greater

trochanter to indicate initial leg length. A stitch or electrocautery mark may suffice to tag the soft tissue at this reference point, i.e. 5 cm. along the ruler (Fig. 3).

The ruler is removed, the hip dislocated and the total hip arthroplasty performed.

Once the trial components or prosthetic implants have been inserted, the legs are realigned as before and the ruler is again inserted over the bone pin. Any change in the level of the ruler along the bone pin will indicate the amount of change in the hip offset (Fig. 4). The closer the ruler is towards the end of the bone pin, the greater the hip offset. Any change in position of the reference mark from its original location along the ruler will determine the relative lengthening or shortening of the leg (Fig. 4).

Fig. 1

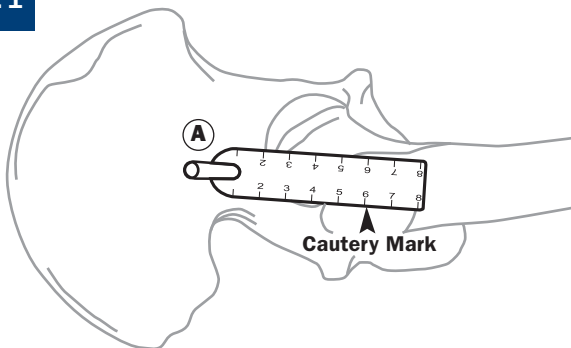


Fig. 2

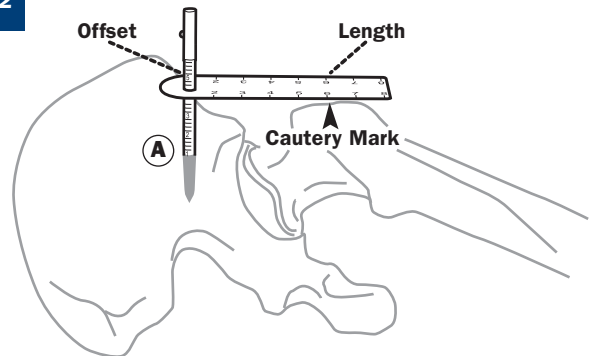


Fig. 3


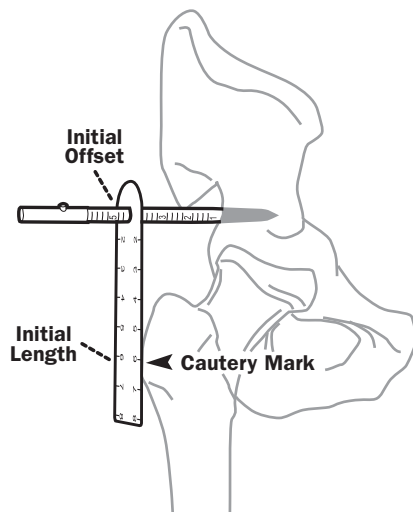


Fig. 4

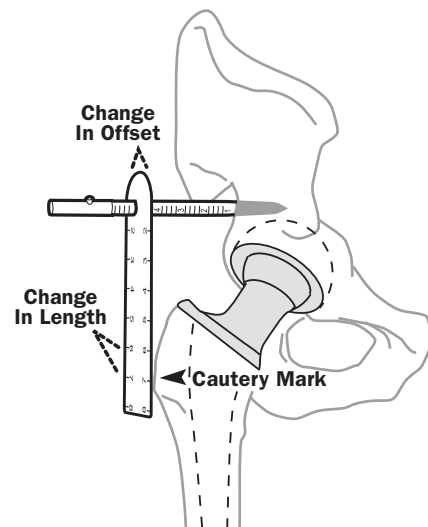

— over —